



Brand Name : AVALANCHE

Product Type: Medical Face mask, 3-Ply with earloop

Ratings: Type IIR

Product code: AV406080 BLUE

# **USER INSTRUCTION**

**Intended Use**: disposable medical masks are suitable for medical workers and family workers working in general medical environment.

Shelf Life: 3 years

## **Storage Condition:**

- 1. Store in a clean, dry environment, away from contaminated and damp places.
- 2. In the process of transportation should be protected from impact and pressure, and try to avoid humid environment

## Use Instruction:

- 1. Open the product package, put the nose strip side of the mask upward, and the rubber bands at both ends are hung on both ears;
- 2. Press the bridge of the nose to fit the bridge of the nose, and then press the nose clip to pull the lower end of the mask to the jaw;
- 3. Adjust the mask wearing effect to cover the bridge of the nose to the jaw, so as to obtain the best protective effect.

## Caution:

- 1. This product is disposable and cannot be reused;
- 2. The product should not be used after expiration due to mildew, yellowing, or damage to the outer package or pollution.
- 3. The product should be placed in a dry place, fire prevention, anti fi moisture

#### Disposal

Never touch the front of the product when removing. This can lead to gross contamination of hands and the product will become be infected with any viral particles during wear. All products listed are single-use disposable, and should be placed straight into a bin once worn. Never share your product with another, as the risk of infection can be high.

Wash your hands! It is critical that hands are washed thoroughly upon removal of any facial protection.

Manufacturer: Hubei Wanli Protective Products Co., Ltd.

Production address: Tangwan Street, Xiliuhe Town, Xiantao, Hubei, China

**Telephone**: 0728-3227299

# 7x9cm

## AVALANCHE Medical Face mask, 3-Ply with earloop Product Dimension : 17.5 x 9.5 cm Product code: LOT Number : 20201075 Package Quantity: 50 pieces/pack, 2000 pieces/box Inspection Date: Date of Manufacture: 2020-10-20 European Standard: EN14683:2019 + AC:2019 Annex 2019 Type IIR, ISO 22609:2004 The storage temperature is -20C - 40C, the storage humidity is <80%, valid for 3 years in a dry indoor environment. Expire Date: Hubei Wanli Protective Products Co.LTD Yuanshi, Ganhe Town, Xiantao City, Hubei province China Manufacturer: SUNGO Europe B.V. Olympisch stadion 24, 1076DE Amsterdam. The Netherlands EC REP: AVALANCHE. The Netherlands. www.vinyl-handschoenenspecialist.nl Distributed by

## 5x6cm

产品合格证	
产品名称:	VR 次性使用医用口罩
	亩 用1 (10 x 9.5cm(卫生级别为普通级)
成分比例: 🛶 🗞	<b>焙火(4</b> , 无纺布66%
型号: 🛂 🖔 📻	±4±1WL702002
医疗器械生产许可证	鄂金河盖成生产许20180820号
医疗器械注册试验	鄂林公20182142554
医疗器械执行标准	YY 0969-2013
医疗器械技术要求编号:	鄂械注准20182142554
注册地址:	湖北省仙桃市干河办袁市四居委会
生产单位:	湖北万里防护用品有限公司
生产地址:	湖北省仙桃市西流河镇塘湾街
生产批号:	202010075
生产日期:	2020-10-20
有效期:	2023-10-19
检验员:	001
售后电话:	0728-3227299